# Aspina

DO M

The logo for 'ekom' is displayed in a bold, green, sans-serif font, enclosed within a green rounded rectangular border. The background of the entire page features a collage of images related to air conditioning and technology, including a green ceiling-mounted unit, a computer monitor showing a software interface, a network switch, and a building facade. A large, faint watermark of the 'ekom' logo is visible in the upper right quadrant.

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INSTALLATION, OPERATION AND MAINTENANCE MANUAL

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<b>IMPORTANT INFORMATION .....</b>	<b>15</b>
<b>1. WARNINGS.....</b>	<b>15</b>
1.1. CE Marking .....	15
1.2. General warnings .....	15
1.3. General safety warnings.....	15
1.4. Safety warnings regarding the protection against electric current.....	15
1.5. Alert notices and symbols .....	16
<b>2. PRODUCT INFORMATION .....</b>	<b>17</b>
2.1. Intended Use.....	17
2.2. Product Description.....	17
<b>3. SCOPE OF DELIVERY .....</b>	<b>17</b>
<b>4. TECHNICAL DATA .....</b>	<b>18</b>
<b>5. FUNCTION DESCRIPTION .....</b>	<b>18</b>
5.1. Function Description .....	18
5.2. Detailed Description of the Suction Part Function.....	19
<b>ASSEMBLY .....</b>	<b>19</b>
<b>6. STORAGE AND TRANSPORT CONDITIONS .....</b>	<b>19</b>
<b>7. INSTALLATION AND INITIAL START-UP.....</b>	<b>19</b>
7.1. Environmental Conditions .....	19
7.2. Electric Connection .....	20
7.3. Initial Start-up.....	20
<b>USE.....</b>	<b>20</b>
<b>8. OPERATION .....</b>	<b>20</b>
8.1. Switching on the Dental Aspirator .....	20
<b>9. MAINTENANCE INTERVALS - USER / TECHNICIAN.....</b>	<b>21</b>
<b>10. MAINTENANCE, CLEANING AND DISINFECTION .....</b>	<b>22</b>
10.1. Inlet Sieve Cleaning .....	22
10.2. Disinfection of Tubings and Separation Vessel .....	22
10.3. Replacement of the Output Pre-filter .....	22
10.4. Replacement of the Output Filter.....	22
<b>11. PUTTING OUT OF OPERATION.....</b>	<b>23</b>
<b>DISPOSAL.....</b>	<b>23</b>
<b>12. DISPOSAL OF APPLIANCE .....</b>	<b>23</b>
<b>TROUBLESHOOTING .....</b>	<b>23</b>
<b>13. INFORMATION ON REPAIR SERVICE .....</b>	<b>23</b>
<b>14. SOLVING COMMON PROBLEMS .....</b>	<b>23</b>
<b>15. WIRING DIAGRAMS .....</b>	<b>49</b>
<b>16. FUNCTION DIAGRAM .....</b>	<b>51</b>
<b>ENCLOSURE NO. 1.....</b>	<b>52</b>
<b>GUARANTEE .....</b>	<b>53</b>

**IMPORTANT INFORMATION****1. WARNINGS****1.1. CE Marking**

Products marked with **CE** mark of compliance meet safety guidelines of European Union (93/42/EEC).

**1.2. General warnings**

- The installation, operation and maintenance manual is an integral part of the appliance. It is necessary to always keep this document close to the appliance. Strict observance of this manual is a prerequisite for the correct operation of the appliance.
- The safety of operating personnel and failure-free operation of the appliance are ensured only when using the original components of the appliance. It is possible to use accessories and spare parts mentioned in the technical documentation or clearly permitted by the manufacturer.
- When used with non authorized accessories or consumable material, the manufacturer cannot assume responsibility for the safe operation and functionality of the device.
- The Guarantee does not cover damages that originate due to the use of non authorized accessories or consumable material other than those recommended by the manufacturer.
- The manufacturer assumes responsibility regarding safety, reliability and function only if
  - The installation, calibration, amendments, extensions and repairs are made by the manufacturer or his representative or a service organization authorized by the manufacturer,
  - The appliance is used in accordance with the installation, operation and maintenance manual.
- The installation, operation and maintenance manual, at the time of printing, corresponds to the design of the appliance and its state according to the relevant safety and technical standards. The manufacturer reserves all copyrights for the given wiring schemes, methods and names.

**1.3. General safety warnings**

The manufacturer developed and constructed the appliance so that damage would not occur when the appliance is used for its intended purpose. The manufacturer considers it his obligation to describe the following safety measures in order to avoid further damages.

- When operating the appliance, it is necessary to observe laws and regional regulations valid in the place of usage. In order to ensure safe course of works, the operator and user are responsible for the observation of regulations.
- The original packaging should be kept for the possible return of the unit. Only original packaging guarantees an optimal protection of the appliance during transportation. If it would be necessary to return the appliance during warranty period, the manufacturer is not responsible for damages caused by incorrect packing.
- It is necessary that the user ensures the appliance is safe to use prior to usage.
- The user must familiarize himself with the correct operation of appliance.
- If an undesirable event occurs in the operation of appliance, the user is obliged to immediately inform his supplier to this event.
- This product is not intended for use in areas with the risk of explosion.

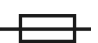
**1.4. Safety warnings regarding the protection against electric current**

- The appliance must only be connected to an appropriate power source that has correct grounding.
- Prior the connecting the compressor, verify whether the mains voltage and frequency specified on the apparatus are in accordance with the local supply.

- Prior to putting into operation, check for possible damages on the appliance and the air connectors. Damaged cables and sockets/plugs must be replaced immediately.
- In the case of a dangerous situation or a technical failure, immediately disconnect the appliance from mains supply.
- During all repairs and maintenance:
  - ensure that the mains plug is removed from the power socket
  - pressure pipes must be air vented
  - pressure must be released from pressure tank.
- This appliance can only be installed only by a qualified expert.

**1.5. Alert notices and symbols**

For your information, the symbols below are used in the installation, operation and maintenance manual, on packaging materials and on the product :

	Information or instructions to prevent any injury to health or material damage to the compressor.
	Alert against dangerous electric voltage.
	Special information regarding the correct use of the appliance and other warnings.
	CE – marking
	Attention! Hot surface.
	Handling mark on packaging – Fragile, handle with care
	Handling mark on packaging – This way up (Vertical position of freight)
	Handling mark on packaging – Protect against humidity
	Handling mark on packaging – Temperature of storage and transport
	Handling mark on packaging – Limited stacking
	Mark on packaging – Recyclable material
	Connection of protective ground wire
	Connector for equipotential connection
	Fuse
	Alternating current
	Danger of biological hazard

## 2. PRODUCT INFORMATION

### 2.1. Intended Use

The dental aspirator ASPINA DO M is a mobile equipment extending the possibilities of procedures provided for by a physician and ensuring the increase of ergonomics at his work. It is suitable for the dental units not equipped with the suction apparatus and a separator. It is intended for the dental practice needs for aspiration, separation and entrapping of the waste into the built-in separation vessel. With regard to its mobility it enables a simple transport between the workplaces and can be used above all where the connection to a sewerage system is impossible.

- The dental aspirator is designed for the operation in dry, ventilated rooms, with the ambient temperature ranging between +5°C up to +40°C, and relative humidity not exceeding the value of 70%.
- The dental aspirator may not be exposed to rain. The apparatus may not be operated in the moist or wet environs. In addition, avoid the use near gases or combustible liquids.
- Other use, or the use exceeding this scope, can not be regarded as the intended use. The manufacturer is not responsible for the resulting damages. The risk will be born exclusively by the operator / user.

### 2.2. Product Description

The mobile dental aspirator ASPINA DO M is built-up on a movable bogie on which is situated a box, lined by a noise damping material. Inside the box there is situated a fan cooled suction aggregate (9) with the electric distribution and the waste entrapping separation vessel (11). In the lower part – under the box – there is situated a silencer with an output filter (14) and a pre-filter (15) ensuring the air filtration from the suction aggregate. In the upper – narrowed part of the aspirator - there is situated a holder for suction tubings (2) equipped with mouthpieces (1), separation automatics and terminal box with fuses. On the lateral part there is situated a main switch (5), over which are placed the indicators for the network (3) and for the state of filling of the separation vessel (4).

## 3. SCOPE OF DELIVERY

- |  |                     |
|--|---------------------|
| • Dental aspirator according to the type:                | DO M                |
| • Instructions for installation, service and maintenance | NP-DO M             |
| • Output pre-filter                                      | 2 pcs No. 025000001 |
| • Fuse T 6,3 A   | 2 pcs No. 038100004 |
| • Antifoam tablets                                       | 2 pcs No. 099000003 |
| • Set of tips  | 1 pcs No. 604011371 |
| • Reduction of saliva ejector                            | 1 pcs No. 062000241 |

#### 4. TECHNICAL DATA

	DO M
Rated voltage / frequency V / Hz	230 / 50; 230 / 60*; 115 / 60*
Rated input of the aspirator VA	520
maximum flowrate : l/min	1100
maximum underpressure kPa	12
Sound level dB(A)	≤48
Mode of operation of the aspirator	permanent S 1
Dimensions of the aspirator mm	565x350x860
Dimensions of the aspirator with carton (h x w x d) mm	950x560x595
Weight of the aspirator kg	36
Weight of the aspirator with carton kg	40
Model according to STN EN 60 601-1 (IEC 601-1)	apparatus type B, class I.

The rated voltage and frequency can be change by special order \*

#### Climatic conditions for the storage and transport

**Temperature** -25 °C up to +55 °C, 24 h up to +70 °C

**Relative air humidity** 10% up to 90 % (without condensation)

#### Climatic conditions for the operation

**Temperature** +5 °C up to 40 °C

**Relative air humidity** up to +70%

### 5. FUNCTION DESCRIPTION

#### 5.1. Function Description

After switching-on the main switch (5) into the position „I“ the network indicator (3) goes on. Taking the suction tubing (6) off the holder (2) will actuate the suction aggregate (9) and the underpressure occurs at the suction mouthpiece (1). After repeated putting the suction tubing into the holder, the suction aggregate turns off. When the separation vessel (11) is filled with waste products, the suction aggregate turns off and the indicator for the separation vessel (4) filling goes on. Then it is necessary to return the suction tubing back into the holder and to empty the separation vessel. During a longer work, mainly with the desalivating mouthpiece, the box temperature may increase; the cooling fan is then automatically actuated. The fan turns off automatically, when the box temperature drops

## 5.2. Detailed Description of the Suction Part Function

The underpressure air flows together with sucked waste products from the oral cavity through the tubing system from the suction mouthpiece (1) at first through the inlet sieve (7), wherein the solid impurities are entrapped. The underpressure air, together with the sucked waste products, free of solid impurities greater than 2 mm, then flows into the separation vessel (11) wherein it is separated from the underpressure air and entrapped into the separation vessel.

The underpressure air flows afterwards into the suction aggregate (9), from where it is blown off through the silencer. Therein the output air passes through the output pre-filter (15) and the bacteriologic output filter (14). After passing the filters, the air free of impurities is blown off into a free space under the dental aspirator.

## ASSEMBLY

## 6. STORAGE AND TRANSPORT CONDITIONS

The dental aspirator is delivered ex-work packed in a transport carton, preventing the apparatus from being damaged during the transport.

**For the transport, always use an original product packing, if possible.  
Transport the dental aspirator in an upright position.**

Protect the aspirator during transport against moisture, impurity or extreme temperatures.  
The aspirator in an original packing can be stored in warm, dry and dustfree rooms.

Keep the packing material for possible future return shipments. If this is not possible, dispose the packing material in an environmental-friendly way. The transport carton can be disposed off as the old paper.

**Mobile dental aspirator may only be transported with emptied separation vessel.  
Always empty the separation vessel contents before any transport.**

## 7. INSTALLATION AND INITIAL START-UP

**Before the initial start-up, remove all locking means serving for fixing the apparatus during the transport.**

**First actuation can be done only by professional specialist.**

### 7.1. Environmental Conditions

- The apparatus may only be installed and operated in dry, well ventilated and dust-free rooms.
- The mobile dental aspirator should be installed with regard to an easy access for the operation and maintenance and good accessible rating plate.
- The apparatus should stand on a plane, sufficiently stable base (be aware of the aspirator weight, see item 4. Technical data).

**Neither the lines for the connection to the electric network, nor the suction tubings may be cranked.**

- To guarantee a trouble-free operation of the aspirator, the room temperature should never drop under +5°C and exceed +40°C. The ideal ambient temperatures are between +10°C up to +25°C.
- Most of the electric energy used by the suction aggregate (9) is converted to heat and given off to the ambient. During a longer lasting work, above all with the desalivating mouthpiece, the box temperature increases over 40°C causing an automatic actuation of the cooling fan. After the room is cooled down under approximately 32°C, the fan turns off again.

### 7.2. Electric Connection

The dental aspirator model 230V is delivered with the safety plug. It is unavoidable to observe the rules of the local electricity works. The network voltage and frequency must be in compliance with the rating plate data.

- In case that the apparatus is fix connected to the supply of electric voltage, a disconnecting device must be situated near the apparatus.
- If the apparatus is connected to the supply of electric voltage by means of a plug, the socket must be good accessible for safety reasons, so that the apparatus can safely be unplugged from the network in case of a danger.
- The network distribution must be protected by maximum 10 A.

### 7.3. Initial Start-up

After unpacking, put the aspirator on the floor, open the door and check whether the separator's cover (10) with sensing electrodes is attached to the separation vessel (11). In case of need, fix the cover to the vessel and then follow the instructions in the Chapter No. 10. Close the door and connect the aspirator's power cord to the socket. Change the main switch (5) over into the position "I", the indicating lamp for the network (3) goes on and the aspirator is ready for the operation.

**The tips are not provided in the sterile status! It is necessary to wash and sterilize the delivered endings before use (see the caution attached to the product - 135°C)!**

**The sterilisation process of tips shall be in accordance with the relevant national legislation.**

## USE

### 8. OPERATION

**The apparatus may only be operated by the trained staff!**

**Disconnect from the network in case of a danger (unplug) ! Hot surfaces of the suction aggregate. Do not touch - Danger of burn.**

#### 8.1. Switching on the Dental Aspirator

Switch on the dental aspirator by turning the power-supply switch (5) in the position „I“. The aspirator is put into operation automatically by taking the suction tubing (6) with a mouthpiece (1) off the holder (2). It works uninterruptedly until both tubings are put on the holders again or until the separation vessel (11) is full (during an uniform operation it is filled within about 6 – 10 hours). Suction tips holder is equipped with vacuum regulation of suction unit depends on dentist needs.

The state of filling of the separation vessel is evaluated by the separation automatics which interrupts the suction by the aggregate (9) and filling of the separation vessel is indicated automatically with the indicator lamp (4) going on. At this point it is necessary to put the tubings with suction mouthpieces into the holder and switch the power-supply switch off. Afterwards open the door by pulling the grips situated on the side walls, uncouple rubber fixing lugs from the separator cover (10) and pull the separator's vessel out. Hang the separator's cover into the holder on the left side (8).

Empty the separation vessel content into the sewer, rinse the vessel with water, connect with the separator's cover in a reversal order. Set the separator into a bowl (12) in the aspirator box (the bowl is detachable). Check the connection of the cover with the vessel, seating of the separation vessel in the bowl and close the door.

**The tips are not delivered in a sterile state!  
Replace used suction mouthpieces on tubings after each patient!**

**The used tips that are designed for multiple use are necessary to be sterilized after each patient !  
The cannulas are to be exchanged after maximum of 400 cycles of sterilisation or after one year based on whichever occurs first.**

**It's forbidden to cover all blow-holes placed on upper sides of unit.**

**Aspirator contains no spare power supply!**

**Before each switch on running unit it's necessary to check if door of box is closed.**

## 9. MAINTENANCE INTERVALS - USER / TECHNICIAN

Maintenance to be done	Chapter	Time interval	To be performed by
Inlet Sieve Cleaning	10.1	per day	user
Disinfection of Tubings and Separation Vessel	10.2	per day	user
Replacement of output pre-filter	10.3	every 3 months	user
Replacement of output filter	10.4	per annum	user
Exchange of suction cannula		Maximum of 400 cycles of sterilisation or once a year *	user
Regular inspection		1x in 2 years	service technician

\* NOTE: the suction end-pieces are exchanged at one of these events, whichever occurs first.

## 10. MAINTENANCE, CLEANING AND DISINFECTION

Repair works exceeding the scope of a general maintenance may only be carried out by a qualified specialist or by the manufacturer's customer service. Use only spare parts and accessories permitted by the manufacturer.

Before making any maintenance, repair or cleaning, it is unavoidable to disconnect the apparatus from the network (unplug).

Following works must be carried out from the hygiene and the correct aspirator function points of view.

### 10.1. Inlet Sieve Cleaning

Solid particles (sucked together with a liquid component and the air) are entrapped in the inlet sieve (7) during the apparatus operation (Enclosure No. 1, Fig. 1) which therefore must be regularly cleaned, by section 9 (though, always at the end of each shift).

The inlet sieve cleaning is possible only after the moisture inside suction tubings (6) and a sieve is reduced by means of the several seconds lasting air suction through tubings taken off the holder (2). Then turn the power-supply switch (5) off in the position „O“ and slightly lift the cover of the inlet sieve. Afterwards pull the sieve out of its place gripping the holder, remove solid particles and clean the sieve. Then put the sieve on the previous place into the holder and carry out the assembly in the contrary manner to the dismantling.

If the suction of amalgam particles is expected, it is necessary to empty the inlet sieve content into a closing vessel and forward it to the collecting centres.

### 10.2. Disinfection of Tubings and Separation Vessel

For the separation vessel (11) disinfection it is necessary to empty the same, rinse with warm water and clean mechanically with warm water and a disinfectant.

The disinfection of internal suction tubings and of the separation vessel should be performed by section 9, though always at the end of a shift by suction of warm water containing a disinfectant having a limited foaming power through both suction tubings (6) separately. The used disinfectant must be approved in compliance with a valid national legislation on usage for disinfected surface and a type of material. For the disinfectant use, follow the manufacturer's instructions. To find out whether the aspirator works correctly, the following works should be carried out in the defined maintenance intervals.

### 10.3. Replacement of the Output Pre-filter

It is necessary to replace the output pre-filter (15) by section 9. Before making any replacements turn the aspirator power-supply switch (5) off. The pre-filter together with the filter (14) are situated on the left lower part of the aspirator's bogie having the symbol .

For its dismantling, catch the filter holder (13) with one hand and with the other hand slew the fixing lugs (16) by 90° by simultaneous pulling down. Remove the filter holder together with the output filter. Then release the output pre-filter from the cavity wherein also the output filter was situated (check also the state of pollution of the output filter).

For the reassembly, set the output filter into the filter holder, put the output pre-filter onto the filter (direct the output pre-filter with the gummed cloth towards the filter) and insert all the assembly into the cavity space from below. Slew the fixing lugs back by 90° so that they engage into the recessed parts of the filter holder.

### 10.4. Replacement of the Output Filter

When the equipment is used regularly, the output filter (14) must be replaced by section 9. The procedure of the filter dismantling and assembly is the same as that for the output pre-filter (15) replacement.

## 11. PUTTING OUT OF OPERATION

When the dental aspirator is not used for a longer time, it is necessary to carry out the cleaning and disinfection of all parts as described in the Articles No. 10.1, 10.2 and then turn the power-supply switch (5) on in the position “I”, take the suction tubings (6) off the holder (2) and let the air intake flow through (for approximately 15-20 min) so that the suction system is fully dried out. Afterwards put the tubings into the holder and turn off the power-supply switch in the position “0”, disconnect the apparatus from the network (unplug) and remove the suction mouthpieces (1) from suction tubings.

## DISPOSAL

### 12. DISPOSAL OF APPLIANCE

- Disconnect the appliance from mains.
- To follow the rules of personal hygiene for works with contaminated material
- Clean device according, section 10
- To separate, label, packing and providing for decontamination of contaminated parts by course of national regulations
- Liquidate dental suction unit according to local effective statutes.

**Inside parts of unit can be contaminated with biological material by reason incorrect using. Before clearing and waste disposal pass on special institution for decontamination.**

## TROUBLESHOOTING

### 13. INFORMATION ON REPAIR SERVICE

Guarantee and post-guarantee repairs are ensured by manufacturer or organizations and repairers denoted by the supplier.

*Warning !*

Manufacturer reserves the right to make changes on the appliance that however will have no impact on substantial characteristics of the appliance.

### 14. SOLVING COMMON PROBLEMS

**Disconnect the appliance from the network before making any interventions therein.**

Works in connection with the removal of failures may only be carried out by a trained stomer service technician.

In case of repair of parts of device which might be contaminated please follow bellow mentioned instruction:

- Disconnect the appliance from mains
- To follow the rules of personal hygiene for works with contaminated material
- Clean device according, section 10
- To separate, label, packing and providing for decontamination of contaminated parts by course of national regulations
- To perform the repair of damaged parts

<b>Failure</b>	<b>Problem and possible reasons</b>	<b>Way of removal</b>
Aspirator does not work	Mains voltage absent	Check mains voltage in the socket, activate a protective element in the electric distribution (fuse, circuit breaker)
	Interrupted supply of el. energy Power-supply switch off	Check the power-supply switch position („I“), switch on the power-supply switch (network indicator goes on)
	Interrupted power cord	Replace the defective part
	Loose clamp of terminal board	Tighten the clamps
	Interrupted motor wiring, defective thermal protection	Replace suction aggregate
	Defective mains fuse of the apparatus	Check the state of apparatus mains fuse, replace defective fuse (fuses 2 x T 6.3 A are situated in the upper, narrowed part of the aspirator)
	Full separation vessel High current consumption	Empty the separation vessel content Defective starting capacitor, replace the capacitor
Aspirator is switching without taking suction tubings off Aspirator is noisy	Suction unit is/was awash by liquid	Suction unit pass on repair to service – keep all rools and norms for work with contamination material Replace the defective part
	Defective control automatics	Replace the defective part
	Defective microswitch in the suction tubings holder	Replace the defective part
	Box door improper closed	Close the box door
	Defective bearings of the suction aggregate	Replace the defective bearing for an identical type
	Incorrectly mounted filter holder	Mount the filter holder into a correct position
	Incorrectly mounted cover of the separation vessel	Mount the separation vessel cover into a correct position
Aspirator efficiency reduced	Suction unit is/was overflowen by liquid.	Suction unit pass on repair to service – keep all rools and norms for work with contamination material
	Heavy polluted output pre-filter	Replace the pre-filter (see Maintenance intervals art.No.11)
	Heavy polluted output filter	Replace the filter (see Maintenance intervals art.No. 11)
	Leaks in the suction system	Check connections, seal untight connections
	Defective control valve in the suction tubings holder	Replace the defective part
	Heavy polluted inlet sieve	Clean the inlet sieve

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## **GUARANTEE**

### **Certificate:**

The Product is manufactured according to production documentation and approved technical conditions. The product is complete and meets all prescribed conditions.

### **Guarantee:**

The Manufacturer provides a guarantee of the product, in duration of 24 months from the selling date, when the terms of guarantee document are followed.

- In the case of a claim for repairs under guarantee, a correctly completed guarantee document and certificate of quality and completeness of product must be submitted together with the product itself.
- During the guarantee period all failures that occur to the product due to production faults will be repaired free of charge.
- The guarantee does not apply to the product damaged during transportation or during incorrect storage.
- The guarantee does not apply to failures due to incorrect operation of the device.
- The guarantee does not apply to failures due to the use of the product for any other purposes other than which are described in the directions for use or as those which have been agreed with the manufacturer.
- The guarantee does not apply to the product when any unauthorised modifications have been made.

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