

MECTRON PROPHYLAXIS HIGHER PERFORMANCE FOR BRIGHT RESULTS



mectron



mectron

MECTRON
PROPHYLAXIS POWDER
INTENSE

250 g e
8.82 oz

BASED ON SODIUM BICARBONATE
LEMON TASTE

mectron

MECTRON
PROPHYLAXIS POWDER
SMOOTH

250 g e
8.82 oz

BASED ON CALCIUM CARBONATE
NEUTRAL TASTE

mectron

MECTRON
PROPHYLAXIS POWDER
SOFT

350 g
12.32 oz

BASED ON SODIUM BICARBONATE
MINT TASTE

mectron

MECTRON
PROPHYLAXIS POWDER
SENSITIVE

380 g
13.44 oz

BASED ON GLYCINE
NEUTRAL TASTE

→ ALWAYS THE BEST CHOICE FOR ALL CLINICAL NEEDS

Supragingival biofilm removal



- INTENSE
- SMOOTH
- SOFT
- SENSITIVE

Removal of extrinsic stains



- INTENSE
- SMOOTH
- SOFT

Subgingival biofilm removal




- SENSITIVE

Periodontal pockets detoxification




- SENSITIVE

Implant maintenance

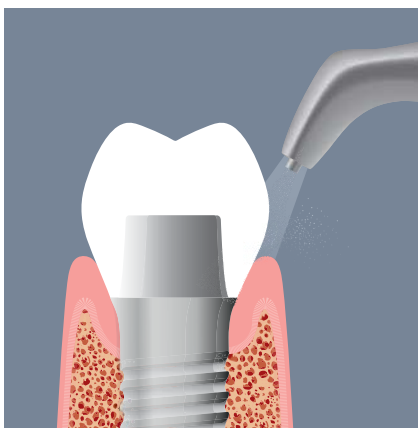

- SENSITIVE

Peri-implantitis treatment

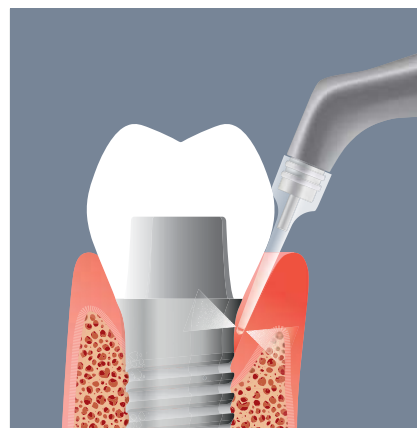


- SENSITIVE

Fixed orthodontic appliances



- SMOOTH
- SOFT
- SENSITIVE

Preparation for whitening

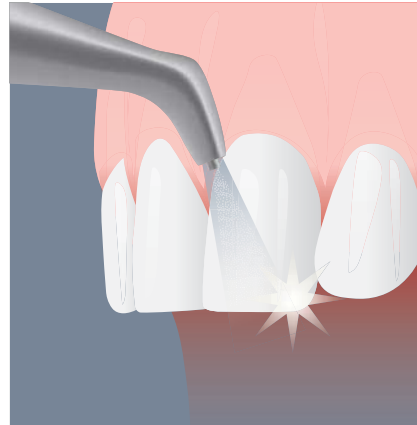

- SMOOTH
- SOFT
- SENSITIVE

Cleaning of restoration materials



- SMOOTH
- SENSITIVE

Cleaning of cavity margins before etching



- INTENSE
- SMOOTH
- SOFT

Cleaning of fissures before sealing



- INTENSE
- SMOOTH
- SOFT
- SENSITIVE

Restorations maintenance



- SMOOTH
- SOFT
- SENSITIVE

→ TAILOR-MADE SOLUTIONS TO SUIT THE NEEDS OF YOUR PATIENT

→ PROPHYLAXIS POWDER INTENSE

effectively removes supragingival biofilm and stubborn extrinsic stains on the enamel surface

INDICATIONS:

- Removal of stains and extrinsic discolorations due to smoke, wine, tea and coloring agents
- Cleaning of fissures and cavities
- Removal of partially mineralized deposits

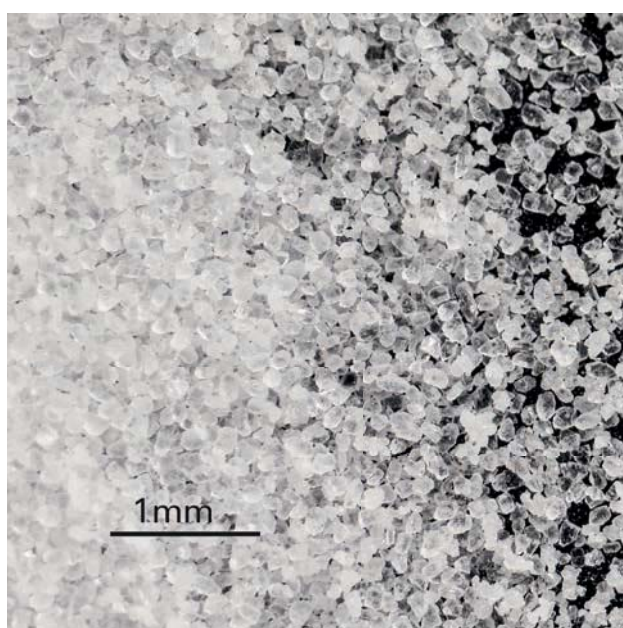


image magnification of 1 mm showing the shape of sodium bicarbonate particles

PROPHYLAXIS POWDER INTENSE

- Sodium bicarbonate-based
- Average particle size less than 120 μm
- Lemon flavour
- Package: 2 bottles of 250 g each
- Ref no. 03140002




PROPHYLAXIS POWDER SMOOTH

gently removes, thanks to its spherical shape, supragingival biofilm while respecting the surrounding tissues

INDICATIONS:

- Teeth cleaning to a shiny finish, while avoiding enamel damage
- Allows better access to the areas difficult to reach
- Aids in reducing the PH acidity in the mouth
- Reduction in aerosol effect and risk of device clogging
- Sodium and silicon (SI) free
- The absence of sodium makes it possible to treat patients following a salt-free diet

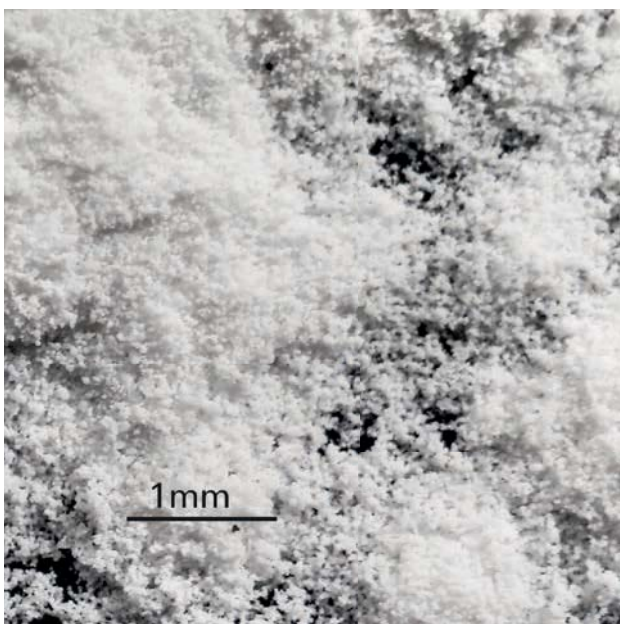

PROPHYLAXIS POWDER SMOOTH

- Calcium carbonate-based
- Average particle size between 45 μm and 75 μm
- Neutral taste
- Package: 4 bottles of 250 g each
- Ref no. 03140010

image magnification of 1 mm showing the calcium carbonate particles with spherical geometry

→ TAILOR-MADE SOLUTIONS TO SUIT THE NEEDS OF YOUR PATIENT

→ PROPHYLAXIS POWDER SOFT*

gently and effectively removes supragingival biofilm and stains on the enamel surface

INDICATIONS:

- Removal of stains and extrinsic discolorations due to smoke, wine, tea and coloring agents
- Cleaning of fissures and cavities
- Cleaning of orthodontic brackets



* Available from autumn 2018

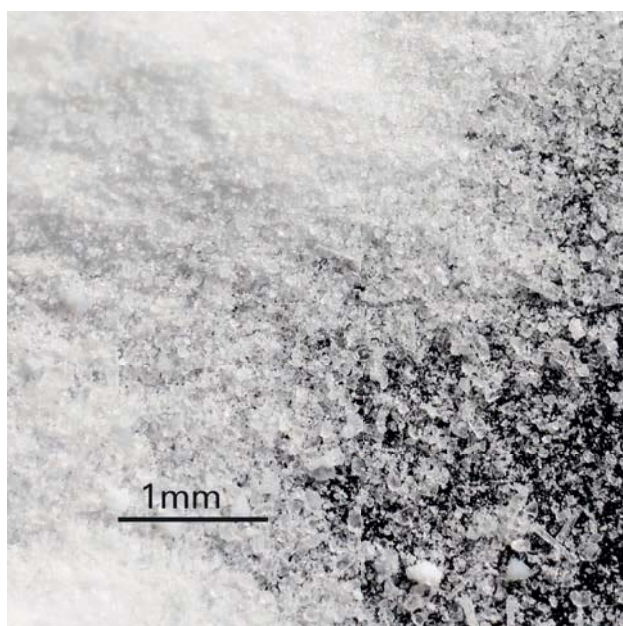


image magnification of 1 mm showing the shape of sodium bicarbonate particles

PROPHYLAXIS POWDER SOFT*

- Sodium bicarbonate-based
- Average particle size 40 μm
- Mint flavour
- Package: 4 bottles of 250 g each
- Ref no. 03140008



→ PROPHYLAXIS POWDER SENSITIVE – GLYCINE POWDER




gently removes supra and subgingival biofilm

INDICATIONS:

- Mild pigmentations and stains removal
- Periodontal pockets detoxification
- Periodontal maintenance
- Implant maintenance and treatment of peri-implantitis
- Orthodontic patient maintenance
- Gentle cleaning of restoration materials and minimally invasive on exposed root surfaces
- Reduction of bacteria colony-forming units (CFU)
- Can be successfully used for the treatment of each type of patient

* Available only in Europe

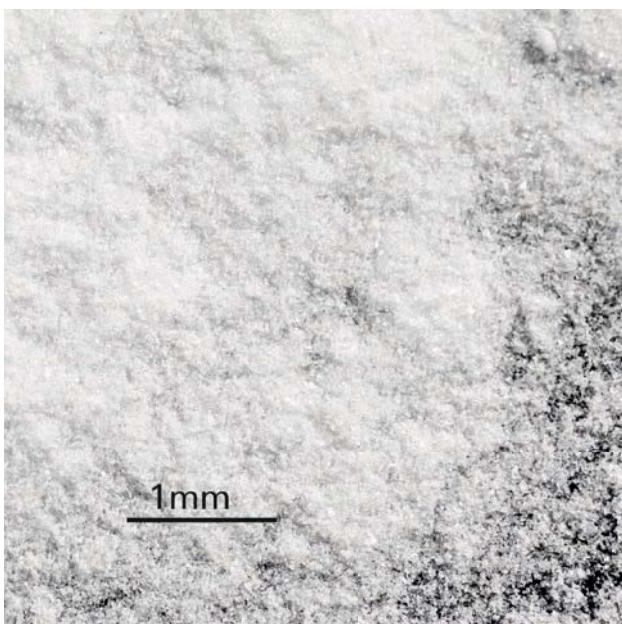

PROPHYLAXIS POWDER SENSITIVE – GLYCINE POWDER*

- Glycine-based
- Average particle size 25 μm
- Sweet taste
- Package: 2 bottles of 160 g each
- Ref. No 03140007

image magnification of 1 mm showing the small sized and minimally invasive glycine particles

THE EVOLUTION OF PROPHYLAXIS



STARJET

One single air-polisher to manage all Mectron powders with different granulometries to perform a complete supra and subgingival prophylaxis treatment

COMBI touch

air-polishing and ultra-gentle ultrasounds in one unit



For more information about the prophylaxis world by Mectron visit the website
www.we-love-prophylaxis.com